

Description

Pro Strips ATTAK is a multiple biothreat detection kit designed to detect up to 5 separate threats using just one small sample. Results can be achieved in as little as 3 minutes.

Important Information

Please review entire kit instructions prior to use. Use only the materials supplied in this kit!

Always use proper PPE or refer to your organization's written protocol.

Humidity levels above 80% can slow results. Do not use in direct rain.

Device works best in temperatures between 38 and 90 degrees fahrenheit or 4 to 32 degrees celsius.

Maximum temperature range:
34 to 120 degrees fahrenheit
1 to 49 degrees celsius.


DO NOT open inner white pouch until you are ready to perform the evaluation.

Contact Us


Advnt Biotechnologies LLC. USA
Toll free 888-223-3269
Emergency information 24/7
602-603-1535
Technical info can also be found at
www.advnt.org

Pro Strips, ATTAK and their logos are registered trademarks of Advnt Biotechnologies 2009


Test Kit Equipment

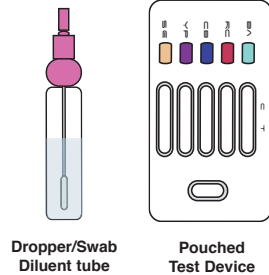


1

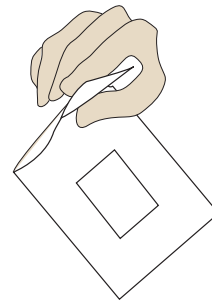


2

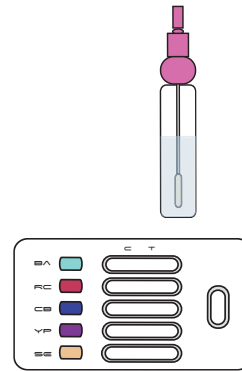


3

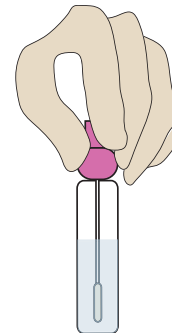

4

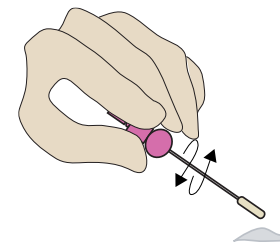

5

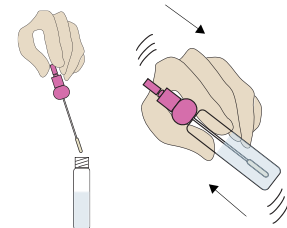

Tear open the inner white pouch to remove the test device. Place the device and the diluent tube on a flat, horizontal surface in front of you.

The test device should be placed in front of you with the text to your left as seen above.

Your instructor will provide a substance necessary to run the test.

Unscrew the cap from the diluent tube. While removing the swab, rub the foam tip against the inside walls to extract excess liquid then remove the swab completely.


Place the foam swab into the powder or liquid supplied by your instructor. Swirl the swab several times in the substance to collect as much sample as possible. Coating the entire swab will provide the best results.

Return the swab back to the diluent tube and tighten the cap. Gently shake the tube for approximately 5 seconds.

MOVE TO STEP 6 ON THE BACK PAGE


Rev. 2 11/08

6

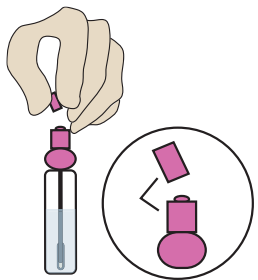

Remove the clear cap from the tip of the diluent tube. Break off the tip and discard. The tube now becomes a dropper for dispensing the sample onto the test device

7

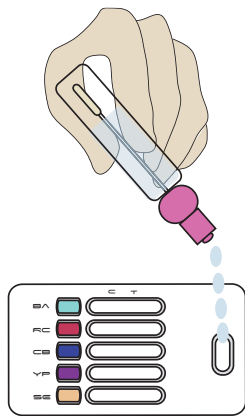


Invert the tube to a 45 degree angle and VERY SLOWLY add 10 to 11 drops to the sample well. DO NOT add more than 11 drops. Return the clear cap back to the dropper and set it to the side.

8




Set a timer and allow a maximum of 15 minutes before interpreting the results. You may see results in as little as 3 minutes. DO NOT interpret results after 30 minutes

9

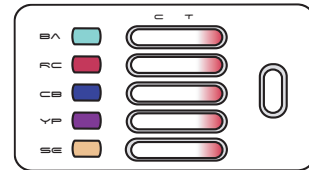


As the test begins to work, a reddish color will move across the five (5) result windows. Lines should always appear at the control line (C) This is an indication that the test is working correctly.

10

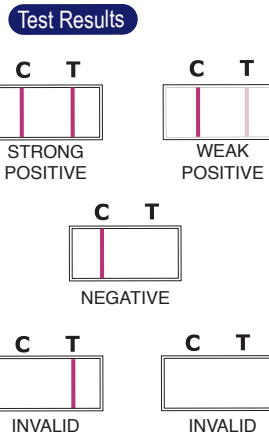

The windows above represent potential results. A weak positive, regardless of its intensity, should be considered positive. Invalids are rare but occur when tests are out-dated or exposed to extreme weather and should be re-run.

11

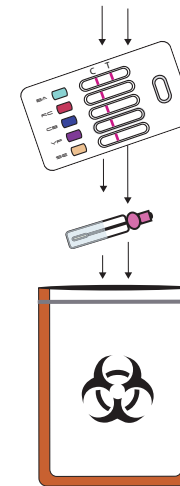

With your results collected, return all items back to the original foil pouch and zip lock shut. All items in this kit are non-toxic and can be safely disposed in the normal refuse.

PRO STRIPS
 Instruction Sheet
 ATTAK™
 PS-A5

Advanced Tactical Threat Assessment Kit